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After use

The instruments need to be cleaned, disinfected and sterilized after every application. For cleaning and disinfection the products are disassembled as far as possible. With the aid of a tweezer the rubber rings on the shank are now removed and disposed. Then, the instruments can be reprocessed. After completed cleaning and disinfection, preferably completely mechanical, a new rubber ring (Art.-No. 2151) is mounted on the shank and the trephine working part is again attached and fixed to the shank by a right-hand turn. Then, the instruments can be sterilized according to the instructions for processing. For a proper processing of the articles please note the instructions for processing (cleaning, disinfection and sterilization) of medical devices from Hager & Meisinger GmbH.

Contraindications

Basically, general medical as well as local, absolute and relative contraindications for dental surgical procedures must be considered.

Absolute contraindications

- Dentoalveolar growth which has not come to an end (exception: cases in which no dentoalveolar growth can be expected, e.g. ectodermal dysplasia)
- Active infections as well as pathological processes
- Insufficient bone supply (quality / quantity) of the donor site as well as insufficient bone quality of the donor region

Relative contraindications

- Diseases that affect bone metabolism
- Drug and alcohol abuse
- Lack of cooperation of the patient
- Poor circulation
- Hard work or active sports
- Mental condition, which can lead to disregard of the medical order
- Highly atrophic jaw

IMPORTANT: Attention must be paid to protecting anatomical structures (safety clearance at least 2 mm) as well as the gradient of the adjacent teeth / tooth roots (risk of damage, infection / dehiscences).

CAUTION: For all cutting and drilling procedures (transplant removal / other work), the following applies: In order to reduce the risk of overheating the bone and necrosis formation along with it, the respective instrument has to be used intermittently using little pressure and continuous cooling using a sterile physiological saline solution.

It should be avoided to trepan the lingual cortical bone as damage to the soft tissue and vessels of the mouth base can lead to serious complications. The extracted bone graft should be used as quickly as possible in the recipient site or stored in blood or isotonic saline solution. It is recommended to fill larger defects after bone extraction. In the donor site, a tension-free and complete wound closure must follow quickly.

General instructions

Please follow general application and safety instructions for MEISINGER products in the medical area and also the advice for processing (cleaning, disinfection and sterilisation) of medical devices from Hager & Meisinger GmbH.

Hager & Meisinger GmbH
Hansemannstr. 10
41468 Neuss • Germany
Tel. +49 2131 2012-0
Fax: +49 2131 2012-222
Email: info@meisinger.de
Internet: www.meisinger.de

Distributed by: **Meisinger USA, L.L.C.**
10150 E. Easter Avenue
Centennial, Colorado 80112 • USA
Phone: +1 (303) 268-5400
Fax: +1 (303) 268-5407
Email: info@meisingerusa.com
Internet: www.meisingerusa.com

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